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Manufacturer:

First Scientific Dental GmbH, 25335 Elmshorn, Germany

EXCIBOND SE

Instructions for Use

Self-etching light cure adhesive

EXCIBOND SE is a simple to use **self-etching light curing one component adhesive**. It ensures a strong bonding of **composites to enamel and dentin**.

EXCIBOND SE is compatible with all current brands of visible light cure composite restorative materials.

Indications

Adhesive for:

- Composite based restorations with light cure composites

Side effects

In singular cases, EXCIBOND SE may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of EXCIBOND SE. Neither store the material in proximity of eugenol containing products, nor let the material allow coming into contact with materials containing eugenol.

Application

1. Isolation

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

Clean the tooth with flour of pumice and water prior to preparation. Prepare the cavity with minimal tooth reduction. Margins should have a slight (**0.5 - 1.0 mm**) bevel placed in the enamel to increase the surface area for greater bond strength.

3. Pulp Protection

For pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

4. Application of EXCIBOND SE

Apply EXCIBOND SE with a brush onto the **slightly moist enamel and dentin surfaces** for at least 15 seconds with agitation.

The material should build a homogeneous layer. Air thin gently for 5-10 seconds to remove the volatile components and to disperse the adhesive. Then light cure with a suitable dental light unit for 20 seconds before placement of a composite.

5. Restorative Placement

Apply the restorative material according to the instructions of the manufacturer.

Best results are obtained with application of a thin layer of a light cure flowable composite followed by the application of a moldable composite. Light cure each composite layer separately according to the corresponding user instructions.

Warnings

- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.

Storage

Do not store above 25 °C (77 °F)! Avoid storage in direct sunlight. Do not use after expiration date.

Warranty

First Scientific Dental GmbH warrants this product will be free from defects in material and manufacture. First Scientific Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and First Scientific Dental GmbH's sole obligation shall be repair or replacement of the First Scientific Dental GmbH product.

Limitation of Liability

Except where prohibited by law, First Scientific Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental use only!

Manufacturer:

First Scientific Dental GmbH, 25335 Elmshorn, Germany

CE 0482